















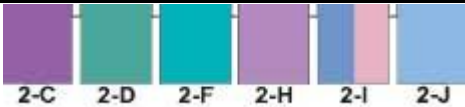
## PenTest – D4D (MD-1)

Please read instruction carefully before use.

①		Take all contents out of the foil pouch.	⑤		Hold the cap with your finger and shake well for 10 seconds.
②		Take the (#633) collection paper. Peel away the protection silicon cover. Hold it at the edge. <b>Do not touch the exposed surface to avoid pre-contamination.</b>	⑥		Remove cap, pointing the tube away from your body. Point the tube down and apply few drops on the paper to cover sampling area.
③		Wipe the suspected area or touch the suspected substance with the test paper.	⑦		Check and compare results carefully with the color chart. The reaction for PCP and Methamphetamines may take up to 90 seconds to fully develop. Discard the test ingredients.
④		Holding the tube upright, squeeze the plastic tube to brake the inside ampoule.  <b>Do not squeeze drug testing tube again over broken glass.</b>	Only color similar to the color chart should be considered as positive results. Each pouch contains two test papers to be used if needed. Only use the collection paper provided with the Pen Test.  <b>Caution: Keep out of reach of children. In case of eye or skin contact, flush well with water.</b>		

## PenTest – C&H (MD-2)

Please read instruction carefully before use.

①		Take all Pen Test drug MD-2 test kit content out of the foil pouch.	⑤		Hold the cap with your finger and shake well for 10 seconds.
②		Take the collection paper from the drug testing Pen Test. Peel away the protection silicon cover. Hold it at the edge. <b>Do not touch the exposed surface to avoid pre-contamination.</b>	⑥		Remove cap, pointing the tube away from your body. Point the tube down and apply few drops on the paper to cover sampling area.
③		Wipe the suspected for the illegal drugs residue area, or touch suspicious substance with the test paper.	⑦		Observe for an immediate appearance of color spot and carefully compare with the color chart below.
④		Holding the tube upright, squeeze the plastic tube to brake the inside ampoule. <b>Do not squeeze drug testing tube again over broken glass.</b>	 <p style="text-align: right; margin-right: 20px;">Where colors matched to the</p> <p>following drugs:                  2-C Methamphetamines or LSD or Efedrine                  2-D Ketamine or Buprenorphine                  2-F Cocaine                  2-H Phenobarbital                  2-I Pure Heroin or Methadone                  2-J Cocaine Base</p>		

Only an immediate color reaction should be considered as positive drug test results. Only colors similar to the color chart should be considered as positive testing results. For each drugs test a new collection paper must be used. Use the collection paper enclosed in each pouch. **Caution: Keep out of reach of children. In case of eye or skin contact, flush well with water.**