



INSTANT-VIEW[®] Multi-Drug of Abuse Urine Tests

TEST PROCEDURE

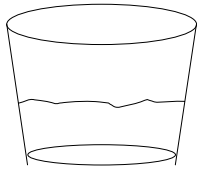


1. Obtain a fresh urine specimen in a clean and dry container.

Test specimen within 8 hours of sample collection.
Do not use urine samples exhibiting visible precipitates/particles.

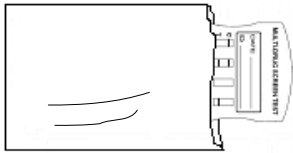

2. Immediately before testing, open the pouch and remove the device from the pouch.

Ensure test device reaches room temperature before opening the package.

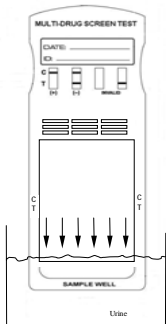

3. Dip the sample well end of the device into the urine sample for 10 seconds or longer.

You must immerse the sample well in the urine specimen completely and keep the tip of the arrows in the device's window above the specimen surface. See picture to the left.



4-7 min.

4. Read results at 4-7 minutes.
Do not read results after 7 minutes.

5. Preliminary positive results must be confirmed by a more specific method (see Instructions for Use)

QUALITY CONTROL


External controls (positive and negative) should be run with each new lot of tests received, each new shipment, each new operator and monthly to determine that tests are working properly.

INTERPRETATION OF RESULTS

Read each test independently. Do not compare color intensity of test lines with each other or with control lines.

PRELIMINARY POSITIVE:


Only a C-line appears

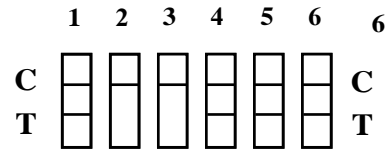

Preliminary Positive for test 2 and test 3

NEGATIVE:

Both C-line and T-line* appear.

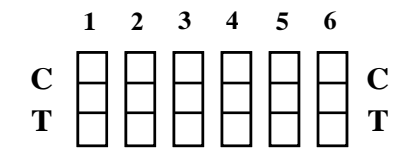


Negative for all 6 drugs

***Even a faint T-Line is negative**

INVALID:

If no lines, or only a T-Line, appear.

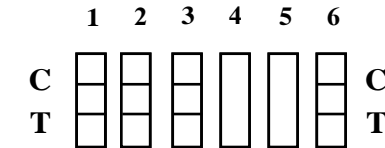

Invalid for test 4 and test 5

If invalid results are obtained, do not report the results. Repeat the test with a new device.

Note that:

1. Substances in urine, including structurally related compounds, can interfere with test results (see Instructions for Use)
2. A positive result does not always mean that a person took illegal drugs. Amphetamine, for example, is in numerous prescribed medicines.

CONFIRMATION OF POSITIVE RESULTS

Preliminary positive results must be confirmed by sending the sample to an accredited laboratory for testing (see Instructions for Use)

To order confirmation kits for this device call Customer Service (310) 997-2846 during weekdays from 9AM to 5PM Pacific Time.

For additional information on confirmation testing call technical service at (310) 863-2410