

# LSD PENTEST INSTRUCTIONS



Open package: tear off at dimple.  
Remove contents



Holding the tube upright, crush both ampoules (press on the 'numbers' printed on the tube). Liquid will appear in light red color.

LSD



Test paper #533: fold silicon cover HALF way back and crease (do not touch paper surface under the silicon cover)



Holding cap with finger, shake ampoule for 10 seconds. **Re-shake** before each test.



Place (or swipe) sufficient amount of drug onto test paper for best results.



Remove cap and SLANT the tube downward (NOT vertical). **Wait** for liquid to fill dropper and then squeeze sides to drop liquid

1. A positive must have the color shown on the color chart, regardless if other colors are present
2. A PenTest has sufficient liquid for at least 2 tests. Once ampoules are broken, the test should be used immediately.
3. Replace cap after using. Store out of direct sunlight.
4. **Caution:** keep out of reach of children. In case of contact with eye, flush well with water